
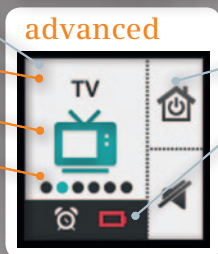


Quick-Fit Guide easyPocket •basic / •advanced

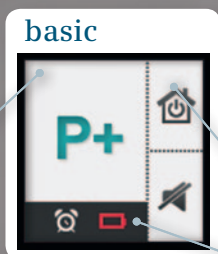

Display

- Indicates change of listening program and volume
- Listening program name
- Listening program icon
- Listening program number

Display

- Status bar (alarm, battery)
- Function of control buttons

Key lock switch

- Slide to lock keys.

Program change button


- Press to change current listening program.
- Program change happens with press even when in standby mode.
- Press to activate display while in standby. Press again to change program.

Volume up button

- Press to increase volume.

Volume down button

- Press to decrease volume.

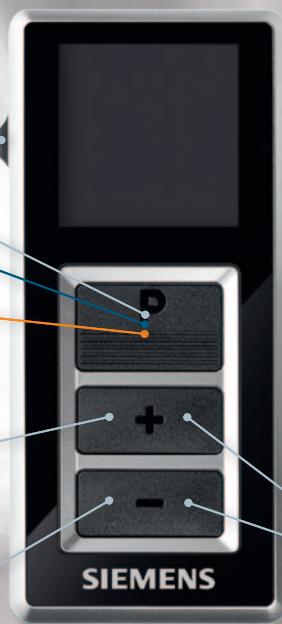

Upper control button [programmable]

- Press to restore hearing instruments to default setting (volume / program) [= Default setting].

Lower control button [programmable]

- Press to turn hearing instruments on / off (mute) [= Default setting].

Volume up + down button (Simultaneously pressed)

- Enter / exit auxiliary menu.

SIEMENS

Quick-Fit Guide easyPocket • basic / • advanced



easyPocket basic

- 1 Select remote control easyPocket.
- 2 Activate easyPocket for the hearing instruments.
(easyPocket must not be hooked up to the computer)

Attention: If you program easyPocket the first time, then "easyPocket basic" becomes an "easyPocket advanced". There is no way to undo this change!

easyPocket advanced

- 1 Select remote control easyPocket.
- 2 Activate easyPocket for the hearing instruments.
- 3 Connect easyPocket with an USB programming cable to your computer.
- 4 Check if the program labels are shown correctly in the preview display. If not, then change the program labels in menu "Program Handling".
- 5 Adjust the configuration of the upper and the lower control button of the easyPocket.
- 6 Adjust the language of the easyPocket auxiliary menu.
- 7 Write your contact data and tick the tick box, if you want them displayed.
- 8 Update easyPocket.